Notice  
**LAMPE D'EXAMEN**

User manual  
**EXAMINATION LAMP**

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[lid-lamps.com](http://lid-lamps.com)

# LAMPE D'EXAMEN

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# EXAMINATION LAMP

Thank you and congratulations for purchasing this examination lamp which has been the object of all our care and attention during its manufacture and quality control. This product will give you full satisfaction for many years. Before using it, you should read this notice carefully.

## Legal compliance

CE marking according to Directives 2014/35/UE, CEM 2014/30/EU and 93/42 CEE and the following harmonized standards: EN 60.598-1, EN 60.598-2-1, EN 60.598-2-6, EN 60.598-2-25, EN 60.601-1, EN 60.601-1-2, EN 60601-2-41, EN 50.581-2-41 (ROHS2).

## Uses

This lamp is specially designed for indoor lighting, it is recommended to use it with our various optional accessories as required.


## Recommendations

Before installing the lamp or carrying out maintenance, always unplug the lamp from the socket. Do not use the examination lamp in a humid atmosphere. Do not close the ventilation openings on the lamp head. Always check that any replacement lamp bulb does not exceed the indicated power rating.

## Maintenance instructions and cleaning

Mounting and connecting the examination lamp should be performed by qualified persons. Before any maintenance, always check that the lamp is unplugged. All cleaning products (*disinfectant, detergent, etc...*) can be used. Difficult stains or spots can be removed with an abrasive cream.

## Installation instructions

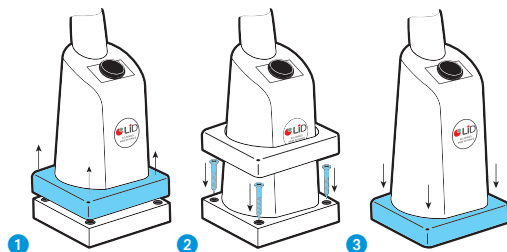
- 1 Pull up the plastic cover from the base of the lamp
- 2 Screw the lamp onto the platform (from the top)
- 3 Replace the plastic cover

## Exclusion of guarantee for lamps equipped with swivel


Do not force the rotation of the lamp head, use the two functions of the swivel (360° and 90°).

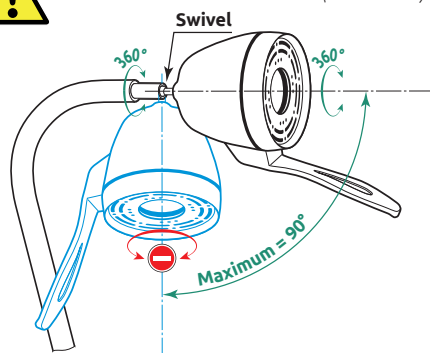## Exclusion of guarantee for flexible necks

Do not bend the flexible neck too close to the lamp base (leave at least 40mm) and do not bend it excessively.

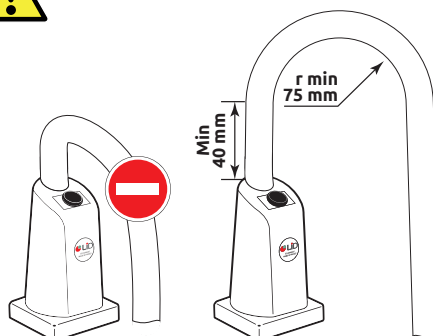If the external power cable of this lamp is damaged, it must be replaced only by the manufacturer, their service agent or a similarly qualified person, in order to avoid any risk.

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